
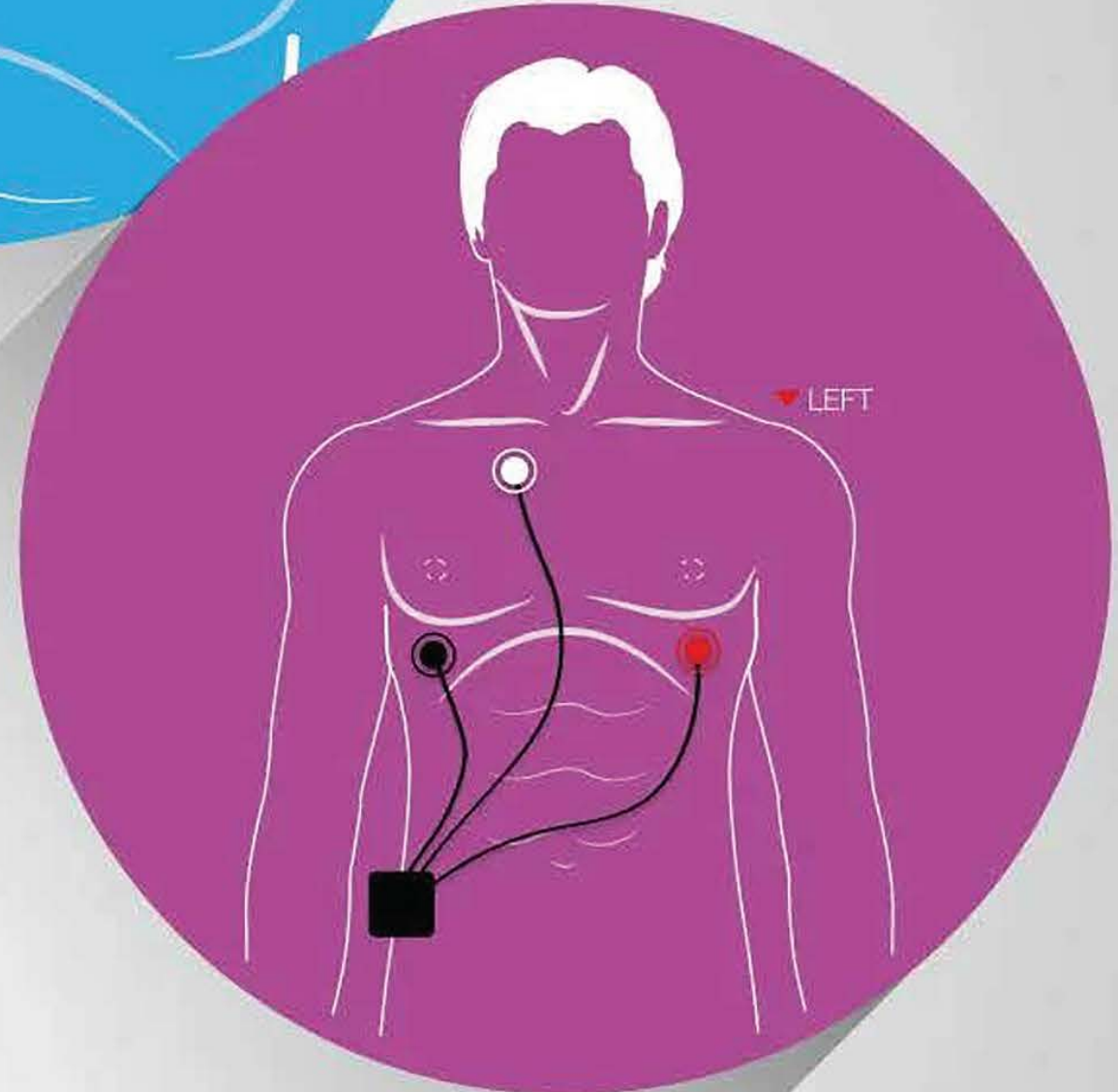
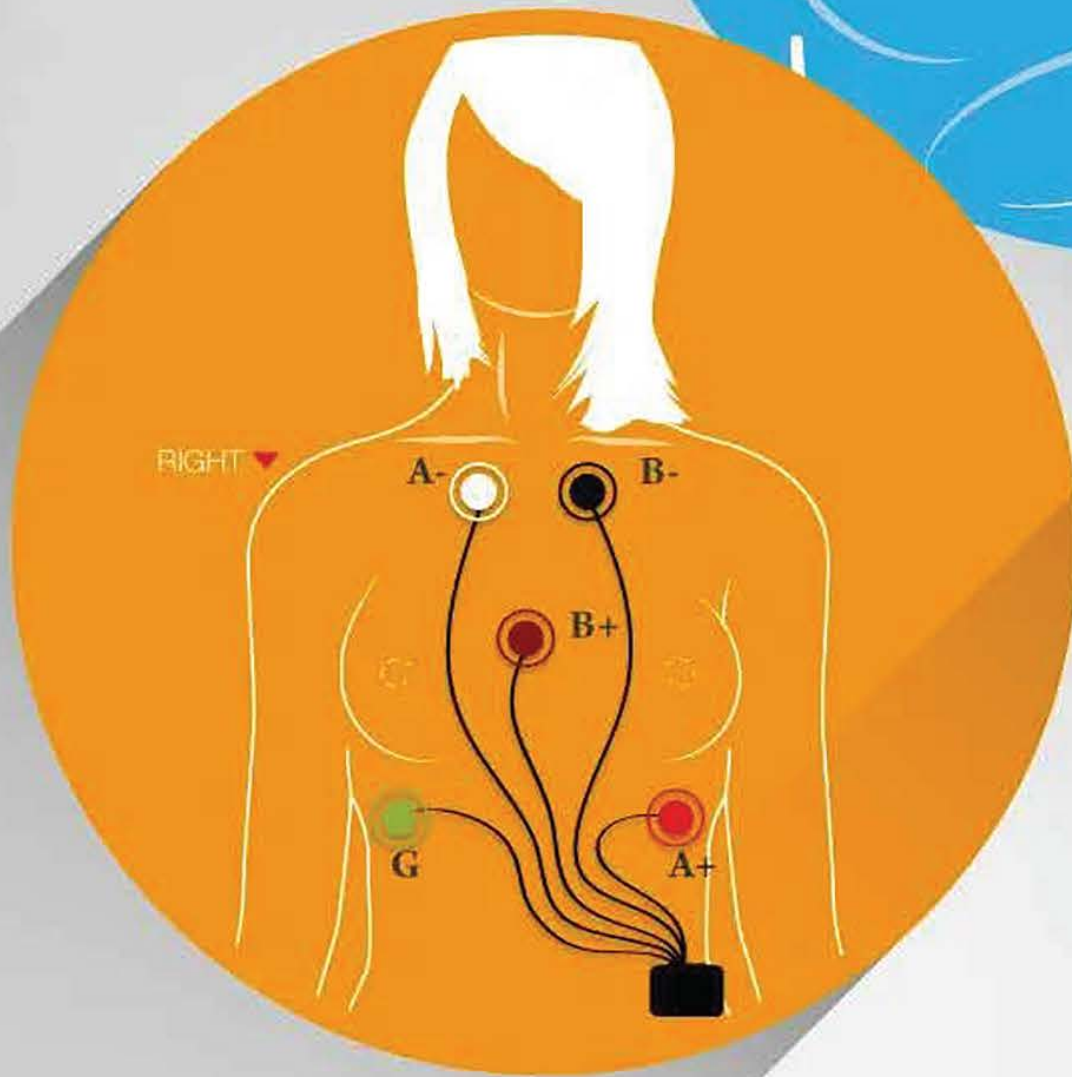


CARDIAC MONITOR PATIENT HOOKUP

TelePatch™

Wireless Dual Channel

- LEFT side of chest
- Flat edge parallel to and one inch below collar bone
- About one inch left of breast bone

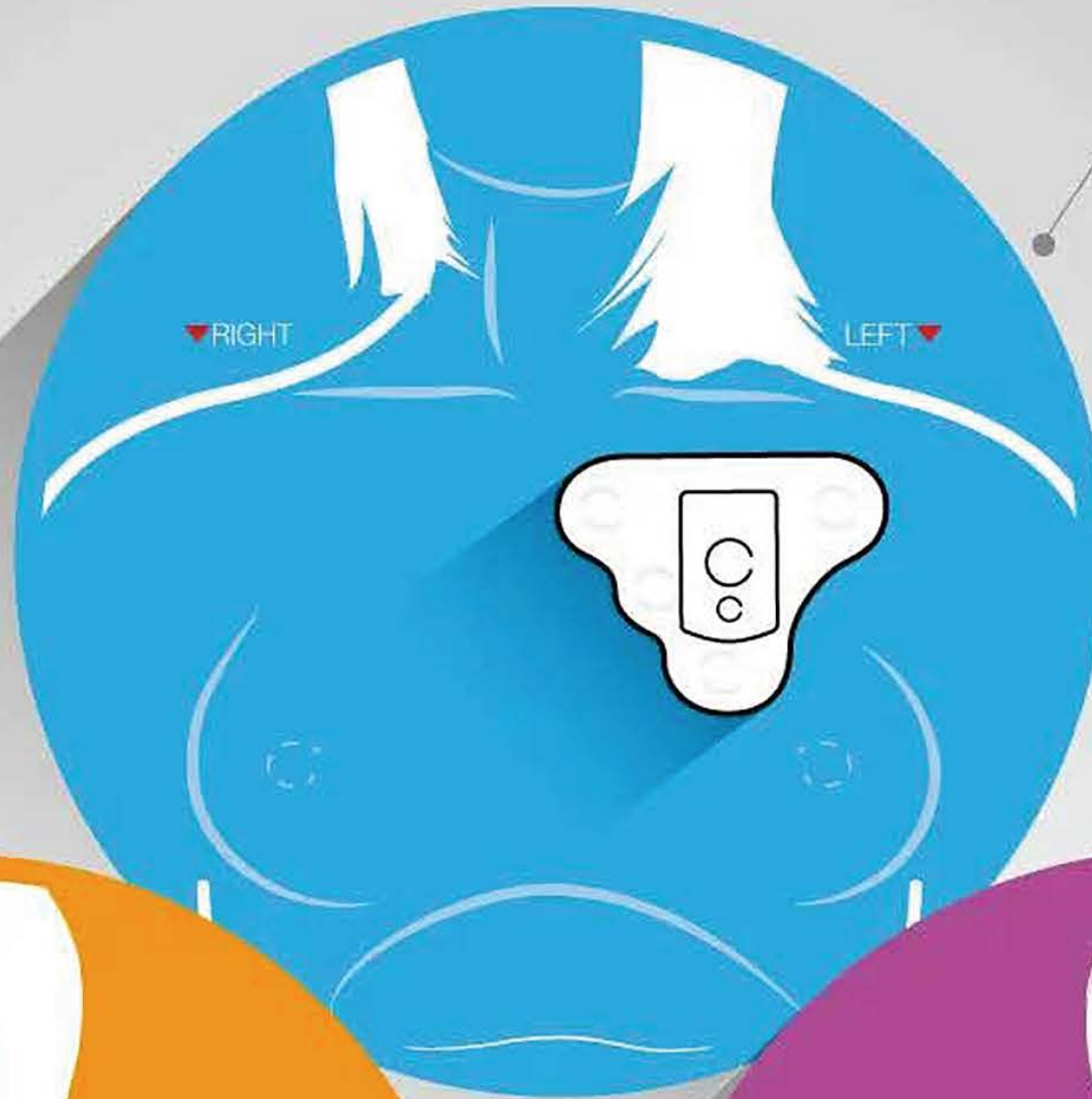

Holter

Wired Configuration

- Place the five colored electrodes as shown in illustration
- White electrode always on upper RIGHT side of chest

Event & MCT


Wired Configuration

- Place the three colored electrodes as shown in illustration
- White electrode always on upper RIGHT side of chest

TelePatch™ Wireless Configuration



EVENT & MCT WIRED CONFIGURATION

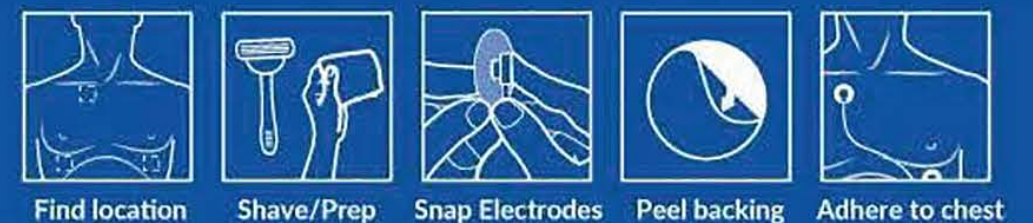

NEED HELP? HAVE QUESTIONS?

Call: (877) 996-5553

Email: CustomerSupport@ReactDx.com

Visit: www.ReactDx.com

For updated product, service and HIPAA information, please visit our website: www.ReactDx.com

PATIENT CENTER

Videos, How-To Guides, FAQs, Troubleshooting
ReactDx.com/Patients

ARRHYTHMIA ACCESS

Your Patient Management Portal
Visit ReactDx.com and click Arrhythmia Access Log In.

